



Cayuga
MEDICAL CENTER

A Member of Cayuga Health System

Laboratory Technical Bulletin


Date: July 10, 2015 Effective Date: July 10, 2015

Title: Transport of Light Sensitive Specimens

Tests: The tests in the following grid must be transported in amber light shielding specimen transport bags.

Place a patient label with the patient’s name and date of birth, or an aliquot label on the exterior of the bag for tracking and accessioning:

Test Name	Test Code
<i>Ascorbic Acid (Vitamin C), Plasma</i>	VITC
<i>Beta-Carotene, Serum</i>	BCARO
<i>Bilirubin, Body Fluid</i>	BFBL
<i>Chlordiazepoxide, Serum</i>	CDP
<i>Methotrexate, Serum</i>	MTHX
<i>Niacin (Vitamin B3)</i>	FNIAC
<i>Riboflavin (Vitamin B2), Plasma</i>	VITB2
<i>Porphyryns, Total, Plasma</i>	PTP
<i>Thiamin (Vitamin B1), Whole Blood</i>	TDP
<i>Vitamin B6 profile (PLP and PA), Plasma</i>	B6PRO

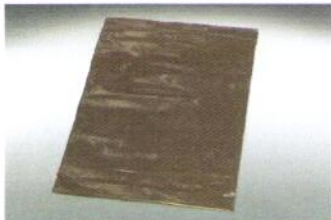

Amber light shielding specimen transport bag.

 _____ Date 7/13/15
Dr. Daniel Sudilovsky, Laboratory Medical Director

 _____ Date 7-10-15
Toni Burger, Laboratory Administrative Director