


# Cayuga Medical Center Laboratory – Blood Culture Fill Line & Labeling

	Bottle(s) to Collect	Volume Required (per bottle)
Routine Draw (Patient with or without Antibiotics)	FAN - Green	5-10 mL
	FAN - Orange	5-10 mL
Pediatric Patient	Yellow	0.5-4 mL
Fungal Culture	Green	5-10 mL

Green & orange bottles: The PRE-PRINTED “FILL” LINE represents the blood culture max fill.

Yellow bottle: The RED LINE represents the blood culture max fill.

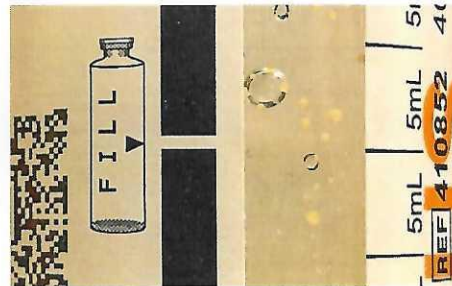

Place the Meditech label on the bottles vertically, with the barcode facing the same direction as the pre-printed bottle barcodes.



Correct label placement



Do NOT cover up or place any labels on the fill window, lot information or 2D barcode.