

Chemistry			Specimen Stability Quick Reference						Hematology								
Test Name	Stability	Container	Test Name	Stability	Container	Test Name	Stability	Container	Test Name	Stability	Container						
Acetaminophen	72 R	SST	FSH	48 R	SST	QFTB	28 Days R	Specialty Tubes	BFCC	24 R	Lav/Sterile Cup						
Albumin	72 R	PST/SST	FT3	48 R	SST	RF	72 R	SST	BFCRY	24 R	Sterile Cont.						
Alcohol	72 R	SST	FT4	48 R	SST	Rubella	48 R	SST	CBCD	24 R	Lavender						
ALP	72 R	PST/SST	Gentamicin	72 R	SST	Salicylate	72 R	SST	CSFCC	1 T	CSF Vial 4						
ALT	72 R	PST/SST	GGT	72 R	SST	Syphilis (RPR)	4 Days	SST	D Dimer	4 R	Blue						
Ammonia	3 ICE/30min T	Pink/Lav	Glucose	72 R	PST/SST/Grey	TBILI	72 R	PST/SST	ESR	12R / 4T	Lavender						
Amylase	72 R	PST/SST	HA1C	7 Days	Lavender	Testosterone	48 R	SST	Fibrinogen	4 R	Blue						
AST	72 R	PST/SST	HDL	72 R	PST/SST	TGAB	48 R	SST	PT/INR	24 R	Blue						
A/V Blood Gas	30 min T	Syringe	Hepatitis	48 R	SST	Theophylline	72 R	SST	PTT	4 R	Blue						
B12	24 R	SST	HIV	72 R	Dedicated Pink	Total T3	48 R	SST	Retic	24 R	Lavender						
BHCG	48 R	PST/SST	Ion Calcium	30 min R	Green	Total T4	24 R	SST	Urinalysis	24R / 1T	Urine						
BHCG reflex	48 R	Red	Insulin	24 R	SST	Total Protein	72 R	PST/SST	Blood Bank								
BNP	24 R	Pink/Lav	K	72 R	PST/SST	TPO	48 R	SST									
BUN	72 R	PST/SST	Lactic Acid	1 T	Grey	Transferrin	72 R	SST	DAT	24 R	Pink						
Calcium	72 R	PST/SST	LDH	72 R	SST	Triglycerides	72 R	PST/SST	Fetal Screen	48 R	Pink						
Carbamazepine	72 R	SST	LDL	72 R	PST/SST	Troponin I	48R / 4T	PST only	Fetal Stain	24 R	Pink						
Carbon Monoxide	7 days R T	Green/Syringe	LH	48 R	SST	TSH	72 R	SST	TS	72 R	Pink						
CEA	48 R	SST	Lipase	72 R	PST/SST	Uric Acid	72 R	PST/SST	*Stabilities in hours unless otherwise noted								
Cholesterol	72 R	PST/SST	Lithium	72 R	SST	Valproic Acid	72 R	SST									
CK	12 R	PST/SST	Lyme	7 Days	SST	Vancomycin	72 R	SST	<table border="1"> <tr> <td>Key</td> <td></td> </tr> <tr> <td>R</td> <td>Refrigerated</td> </tr> <tr> <td>T</td> <td>Room Temp</td> </tr> </table>			Key		R	Refrigerated	T	Room Temp
Key																	
R	Refrigerated																
T	Room Temp																
CKMB	48 R	PST/SST	Magnesium	72 R	PST/SST	Vitamin D	72 R	SST									
Cl	72 R	PST/SST	Myoglobin	24 R	PST/SST	ED Drug Screen	7 Days	Urine									
CO2	72 R	PST/SST	Na	72 R	PST/SST	U Buprenorphine	5 Days	Urine									
Cortisol	48 R	PST/SST	Osmolality	72R / 24R	SST/Urine	U Fentanyl	7 Days	Urine									
Creatinine	72 R	PST/SST	Phenobarbital	72 R	SST	U Hydrocodone	7 Days	Urine									
CRP/CRPHS	72 R	PST/SST	Phenytoin	72 R	SST	U Methadone	7 Days	Urine									
CSF Glucose	5 Days	CSF vial	Phosphorus	72 R	PST/SST	U Oxycodone	7 Days	Urine									
CSF Protein	72 R	CSF vial	Prealbumin	72 R	SST	Urine Chloride	72 R	Urine									
DBILI	72 R	PST/SST	Progesterone	48 R	Red	Ur Creatinine	72 R	Urine									
Digoxin	72 R	SST	Prolactin	48 R	SST	Urine Potassium	72 R	Urine									
Estradiol	48 R	SST/Red	PSA	24 R	SST	Microalbumin	24 R	Urine									
Fe (Iron)	72 R	SST	PTH Intact	4T / 8R	SST	Urine Sodium	72 R	Urine									
Ferritin	48 R	SST	PTH IO	8 T	Pink/Lav	Ur Total Protein	48 R	Urine									
Folate	48 R	SST															

Refer to Cayuga Lab Test Catalog for additional container types and stability times.

MD 9/15/17
Updated 12/2022 KT